Zeichenerklärung/Explanation of symbols/Explication des symboles/Legenda/Explicación de los símbolos/Verklaring van symbolen:

Medizinprodukt/Medical product /Dispositif médical/Dispositivo medico/Producto sanitario/Medisch hulpmiddel

Gebrauchsanweisung lesen und beachten/Read and follow the instructions for use/Lire et respecter le mode d'emploi/Leggere e seguire le istruzioni per l'uso/Leer y seguir las instrucciones de uso/Lees en volg de gebruiksaanwijzing

Nicht wiederverwenden/Do not reuse/Ne pas réutiliser/Non riutilizzare/No reutilizar/Niet opnieuw gebruiken

Vor Sonne schützen/Protect from the sun/Protéger du soleil/Proteggere dal sole/Proteger del sol/Beschermen tegen de zon

Vor Feuchtigkeit schützen / trocken lagern/Protect from moisture / store in a dry place/Protéger de l'humidité / conserver dans un endroit sec/Proteggere dall'umidità / conservare in luogo asciutto/Proteger de la humedad/guardar en un lugar seco/Beschermen tegen vocht / op een droge plaats bewaren

Abb. 1 /fig.1 /fig.1 /ill.1 /ilustr.1 /ill.1

Abb. 2 /fig.2 /fig.2 /ill.2 /ilustr.2 /ill.2

Abb. 3 /fig.3 /fig.3 /ill.3 /ilustr.3 /ill.3

Abb. 4 /fig.4 /fig.4 /ill.4 /ilustr.4 /ill.4

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Kinesiotape // Art.Nr.: 31144

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GB Instruction manual

Kinesio tape // Item no.: 31144

IMPORTANT! THE FOLLOWING INSTRUCTIONS MUST BE READ CAREFULLY AND RETAINED FOR FUTURE REFERENCE KEEP INSTRUCTIONS SAFE!

Intended use:

The reer mama Kinesio tape is an elastic, breathable and water-resistant kinesiology tape. It consists of a cotton material and a hypoallergenic acrylic adhesive. The tape offers support in the lower back area, especially during pregnancy and breastfeeding. The effect of Kinesio tapes has not been scientifically proven; our statements are based on usage reports from customers and midwives, as well as findings from their use in sports.

Safety instructions:

- Please only use the item for its intended purpose.
- Never use the tape on injured or irritated skin.
- The tape is suitable for single use only.
- If you experience skin irritation or an allergic reaction or the tape does not feel good, remove the tape.
- If you have any doubts or health concerns, consult your doctor or midwife before using the product.
- The tape is intended for private use.
- Only use the tape if the packaging is undamaged.

Instructions for use:

- Thoroughly remove oils, lotions and dirt from the skin before use.
- The tape is designed to alleviate pain in the lower back (fig.1)
- Sit on a chair and bend forwards slightly.
- Have the tape applied as follows:
- 1. Remove a few centimetres the backing film in the centre, fold it over and hold the ends of the tape in your hands. (fig.2)
- 2. Pull the tape until half taut and place the centre of the tape on the painful area. (fig.3)
- 2. Remove the remaining backing film and stick on the ends of the tape without pulling. (fig.3)
- 3. To activate the adhesive, rub the tape several times after application. (fig.4)

Removal instructions:

- Wear the tape for a maximum of 7 days.
- Do not tear the tape off the skin abruptly. Carefully peel off in the direction of hair growth. It is easier to remove after or during a shower.

Material: 95% cotton, 5% spandex, hypoallergenic acrylic adhesive

Storage:

We recommend that you store the product out of direct sunlight. Store in a dry place away from the reach of children. Durability and disposal: The tapes have a shelf life of 3 years. The product can be disposed of in the household waste.

Please retain your receipt, the user manual and item number. Warranty claims or claims from promises of guarantee must be processed by the dealer.

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Company
Address

WuXi Wemede Healthcare Products Co., Ltd., NO.1060, Antai
3 Road An Town, Xishan
District, Wuxi 214105 China

SUNGO Europe B.V., Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel, Netherlands

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